# Multi-Mam COMPRESSES

INTENSIVE NIPPLE TREATMENT FOR BREASTFEEDING MOTHERS

## Multi-Mam Compresses - for intensive nipple treatment

- Has a direct soothing effect on sore nipples
- Neutralizes harmful microorganisms. Prevents against infection (mastitis)
- Optimizes the condition of the nipple area
- Gel is natural and harmless when swallowed. Preservative-free!
- Supports the natural healing process

### Application

Multi-Mam Compresses are especially developed for intensive nipple care of breastfeeding mothers. The product is based on the patented 2QR-complex that is unique in its ability to block harmful bacteria in a safe and natural way. Multi-Mam compresses have a direct soothing effect on sore nipples, optimise the condition of the skin, reduce swelling and prevent infection. Each compress is impregnated with bio-active gel. This gel is based on natural plant ingredients that are totally harmless and can be safely swallowed. It is not necessary to remove the gel prior to breastfeeding. The taste it leaves is pleasantly sweet for the baby. The Multi-Mam compress forms a soft and comfortable pad on the nipple and creates an optimal situation for the natural healing process. The thin plastic coating also prevents staining of the bra. Multi-Mam is therefore ideally suited for treatment of nipple problems caused by breastfeeding.

### Directions for use

Open the sachet, unfold the compress and place it with the impregnated side on the nipple. Leave the compress in place as long as desired but for at least 10 minutes, preferably 30 minutes and maximum 1 hour. Repeat treatment at least twice a day until problems are relieved. Multi-Mam Compresses should not be used longer than 30 days consecutive days. In case of persistent or recurrent complaints you should always contact your physician.

### Active ingredients

Galactoarabinan Polyglucuronic Acid Crosspolymer\*, Glycerin (plant origin), Xanthan Gum. \*2QR-complex: patented bio-active polysaccharides. www.2QR.com

### Allergies

Allergies to the ingredients of Multi-Mam Compresses are very rare. When you have doubts, apply a little bit of gel on the inside of the arm. When an itching red spot appears we advise you not to use Multi-Mam Compresses.

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